




# NEURONUP2GO MANUAL (HOME SESSIONS)




## 1. Create a patient

"Patient Management" > "New Patient"

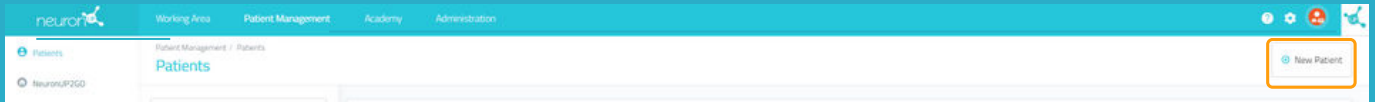

## 2. Create access data to NeuronUP2GO


Patient profile > "Change Access Information"

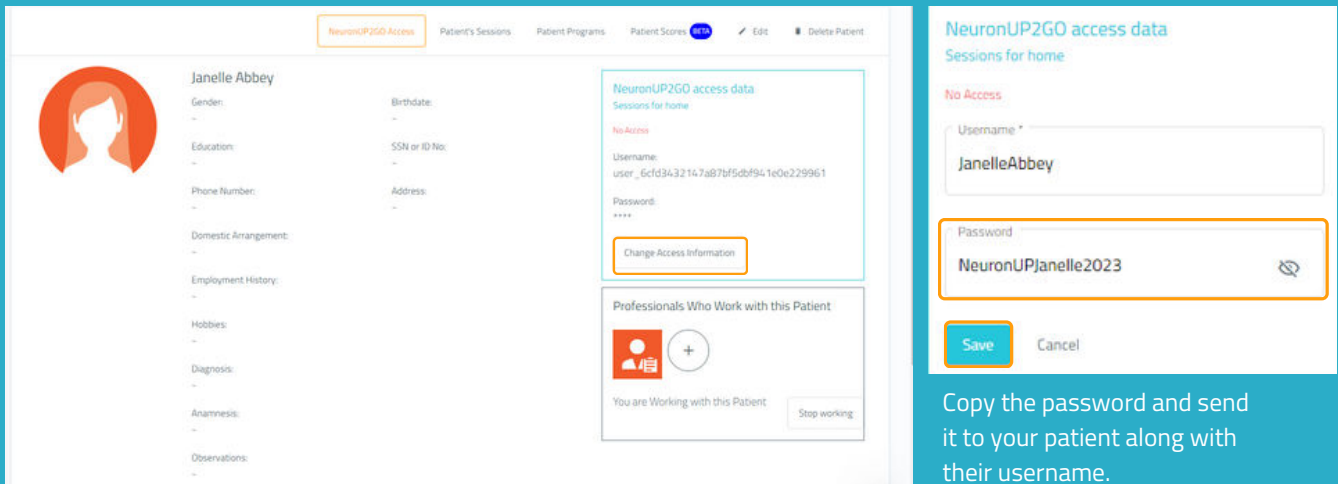


Copy the password and send it to your patient along with their username.

## 3. Assign NeuronUP2GO months

"Patient Management" > "NeuronUP2GO"

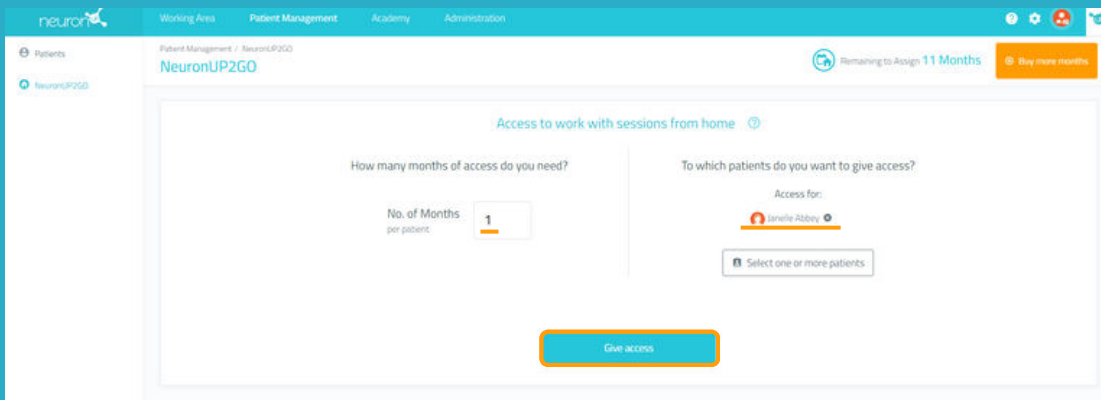

You can assign one or more months to one or more patients.

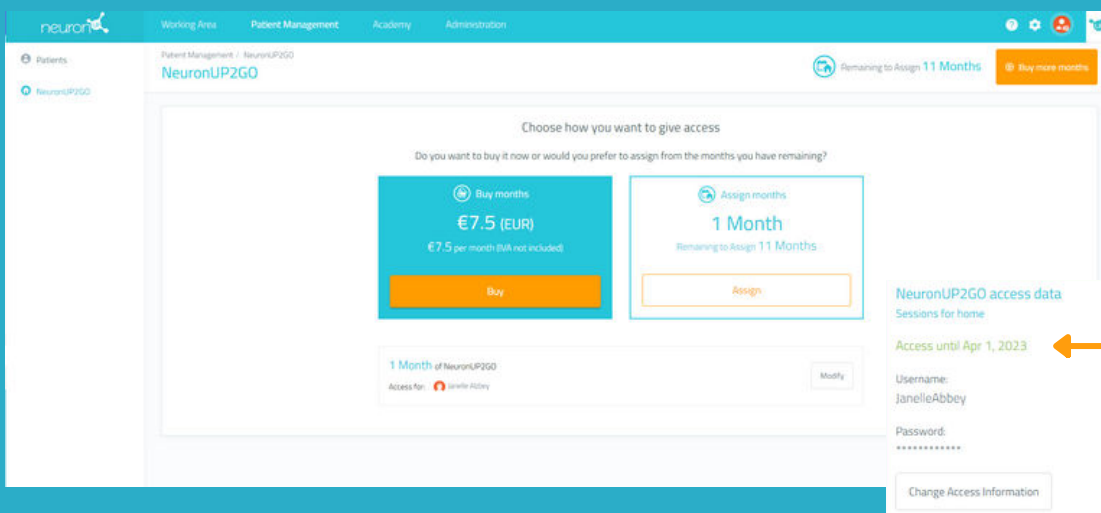

You can buy additional months or assign months you already have. (11 in the example).




Your patient now has remote access (you can see until which day he/she will be able to access).


## 4. Create and assign a home session

Click on "Digital Sessions"

Click on "Assign" in the chosen session.

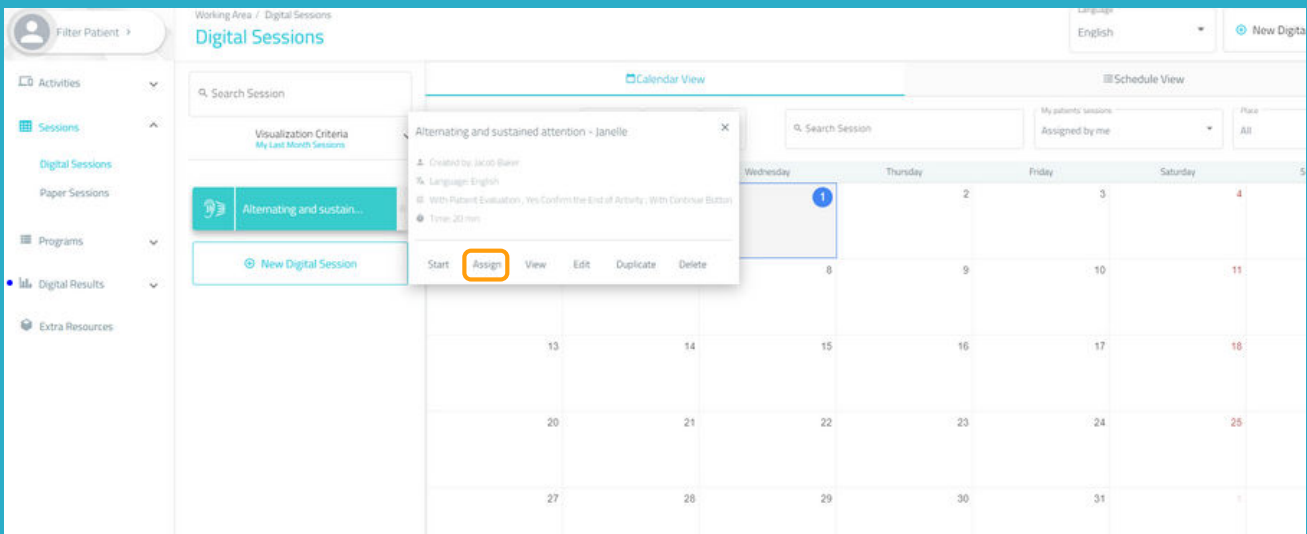

Choose your patient (with active home access).  
Enable the "Session for home" filter.

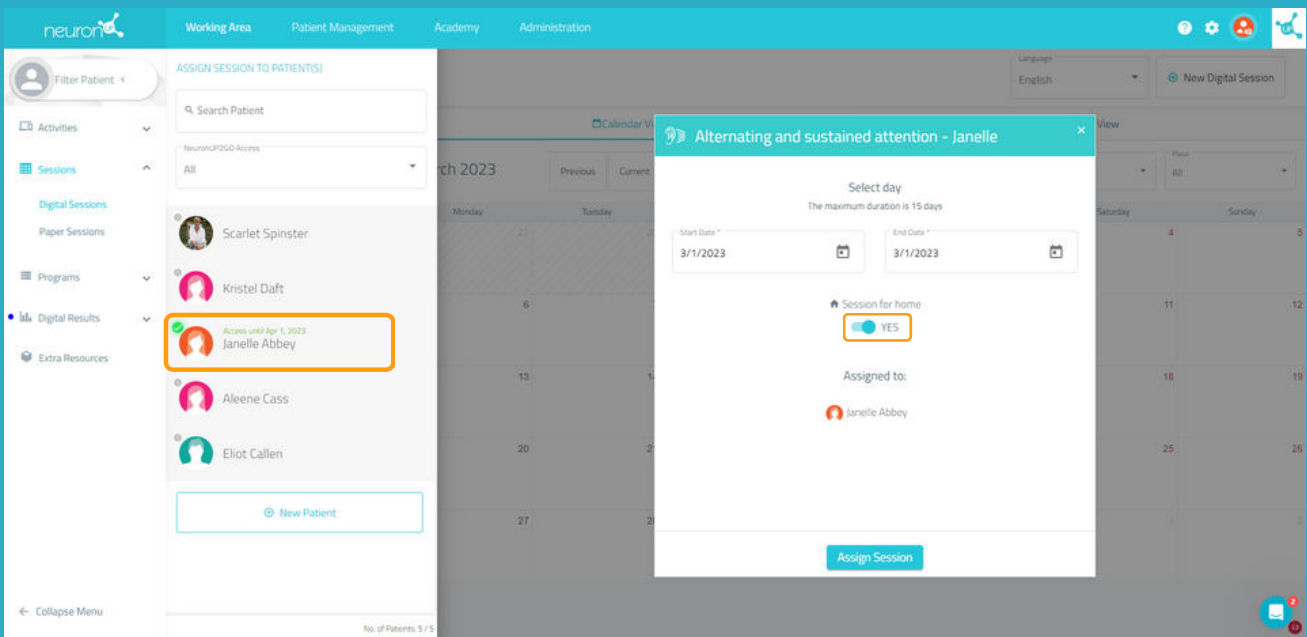

## 5. Follow the progress of the session live

When your patient connects from home and starts the session, you will be able to see their progress thanks to the indicator shown in the image.

